

Dial Vision™

Adjustable Lens Eyeglasses

-6D  +3D
Indicates Lens Power Range

IMPORTANT!

Please read instructions thoroughly before use.

DialVision™ is intended for presbyopia, far and near sightedness only, NOT for astigmatism.

Lens power range: -6.0 to +3.0 Diopters
Frame measurement: 49□22, 141mm
Centration distance: 63mm

Please dispose of device in accordance with local regulations.

* DialVision™ is not intended to replace prescription eyewear or examinations by eye care professionals.

Eye examinations can detect health problems that have no signs or symptoms. Have your eyes checked regularly.

Never disregard or delay getting professional eye health advice because of the purchase, receipt or use of DialVision.

DialVision eyeglasses do not correct astigmatism.

- Should DialVision cause any discomfort, discontinue wear.
- DialVision is designed for short term use.
- DialVision should not be used for high-risk tasks, driving or operating heavy machinery.
- Handle and clean with care.

WARNING: This product contains a chemical known to the State of California to cause birth defects or other reproductive harm.

Instructions For Use

Adjust for Distance:

Begin by Setting Up the DialVision™ Eye Chart:
Using a piece of tape, hang the DialVision eye chart at eye level.

1. Keep a 13 foot distance between yourself and the eye chart for the most accurate results.



This chart should be viewed at a distance of 13 feet



Included Eye Chart

2. Start by turning the dials on both sides of the DialVision frame forward until you hear a click or until you cannot turn the dial any further. (Figure 1)



Figure 1

3. Put on the DialVision glasses and cover your left eye. (Figure 2)

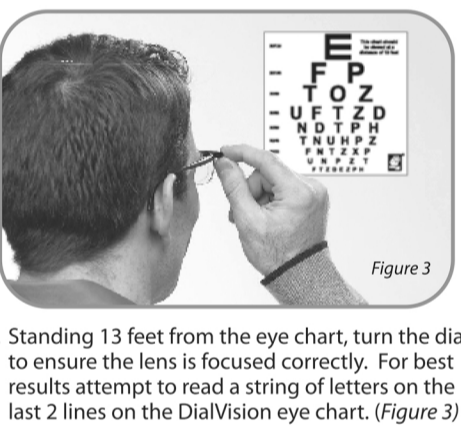


Figure 2

Figure 3

4. Standing 13 feet from the eye chart, turn the dial to ensure the lens is focused correctly. For best results attempt to read a string of letters on the last 2 lines on the DialVision eye chart. (Figure 3)

Note: The average person has 20/20 or 20/25 vision; by successfully reading either of the last 2 lines on the chart from left to right.



Figure 4

Cover Right Eye

5. Now cover your right eye and repeat. (Figure 4)

Adjust for Reading:

1. Place some reading material on a flat surface in front of you so that your hands are free to adjust the glasses for reading. (Figure 1)

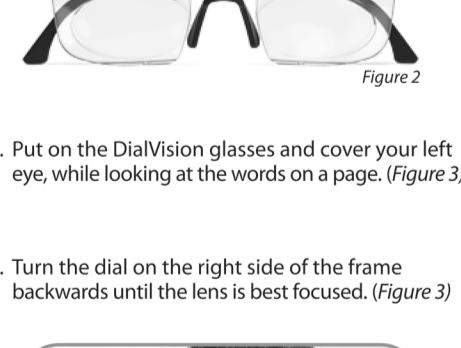


Figure 1

2. Start by turning the dials on both sides of the DialVision frame forward until you hear a click or until you cannot turn the dial any further. (Figure 2)



Figure 2

3. Put on the DialVision glasses and cover your left eye, while looking at the words on a page. (Figure 3)

4. Turn the dial on the right side of the frame backwards until the lens is best focused. (Figure 3)

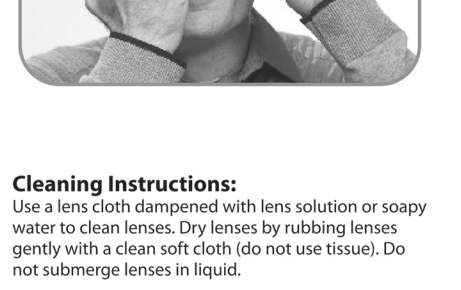


Figure 3

Cover Left Eye

5. Now cover your right eye and repeat. (Figure 4)

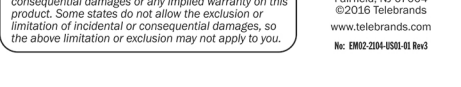


Figure 4

Cover Right Eye

Cleaning Instructions:
Use a lens cloth dampened with lens solution or soapy water to clean lenses. Dry lenses by rubbing lenses gently with a clean soft cloth (do not use tissue). Do not submerge lenses in liquid.

LIMITATION OF LIABILITY
Liability limited to the purchase price of this product. Telebrands Corp. shall not be liable for any incidental or consequential damages or any implied warranty on this product. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

Distributed by:
TEL-Brands
Fairfield, NJ 07004
©2016 Telebrands
www.telebrands.com
No. EM02-2104-US01-01 Rev3